


## DREAMMAPPER – GENERATE REPORT

### INTRODUCTION

This document explains how to generate a report with DreamMapper from a computer or laptop.


### PROCEDURE

1. Go to <https://eu.mydreammapper.com> and login with your username and password.

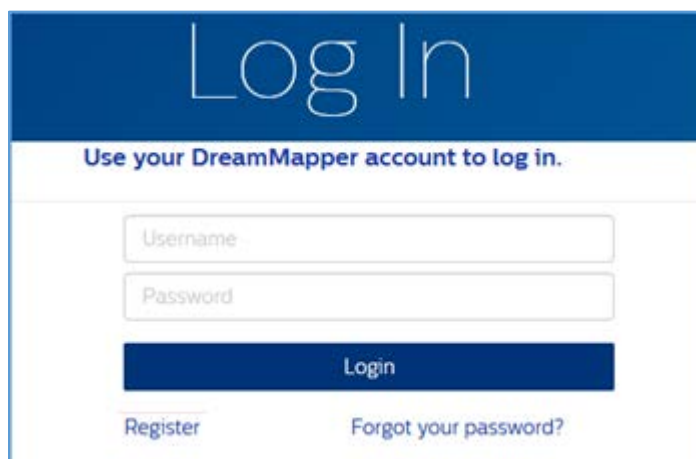

The screenshot shows the 'Log In' page of the DreamMapper application. It features a dark blue header with the text 'Log In' in white. Below the header, there is a white box with the text 'Use your DreamMapper account to log in.' Inside this box, there are two input fields: 'Username' and 'Password'. Below the input fields is a dark blue button labeled 'Login'. At the bottom of the white box, there are two links: 'Register' and 'Forgot your password?'.

2. When you have logged in make sure that your are on the **Sleep** page and click **Show Report**.

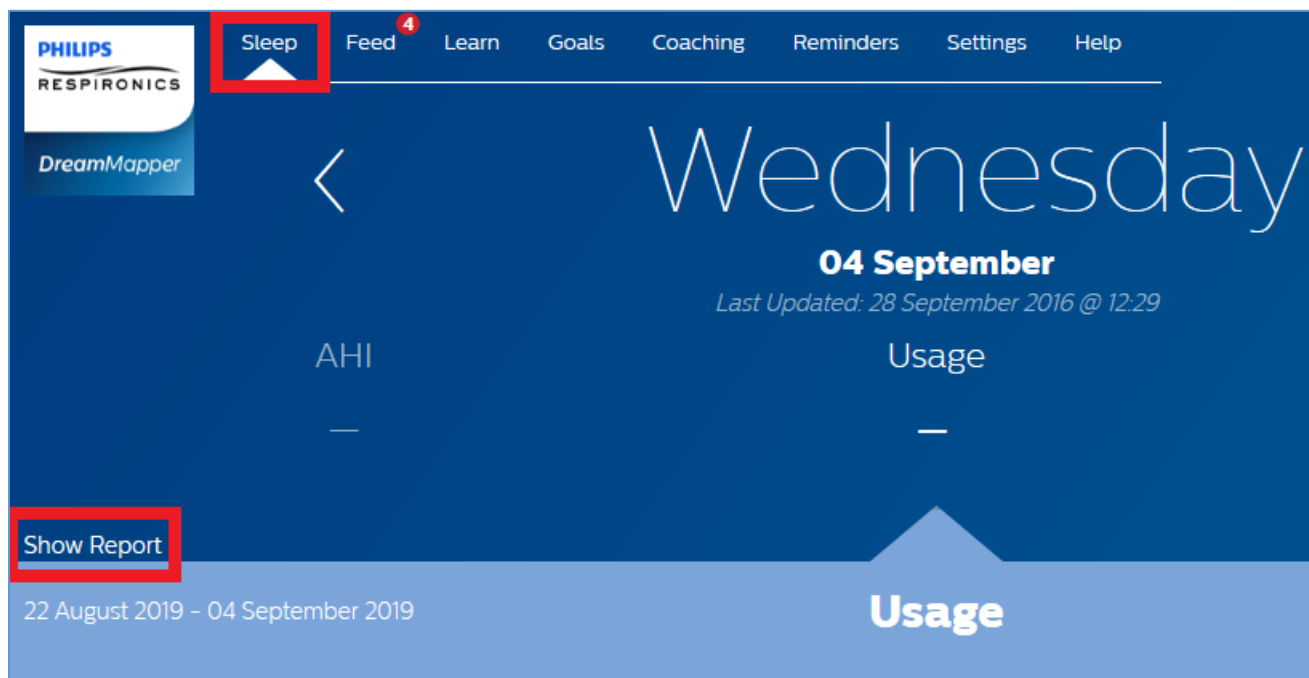

The screenshot shows the 'Sleep' page of the DreamMapper application. The page has a dark blue background. At the top left, there is a Philips Resironics logo and the text 'DreamMapper'. To the right of the logo, there is a navigation menu with the following items: 'Sleep', 'Feed', 'Learn', 'Goals', 'Coaching', 'Reminders', 'Settings', and 'Help'. The 'Sleep' item is highlighted with a red box. Below the navigation menu, there is a large white text 'Wednesday' and '04 September'. Below this, there is a smaller white text 'Last Updated: 28 September 2016 @ 12:29'. Below the date, there are two columns of data: 'AHI' and 'Usage'. Below the data, there is a large white text 'Usage'. At the bottom left, there is a white text '22 August 2019 - 04 September 2019'. At the bottom right, there is a large white text 'Usage'. A red box highlights the 'Show Report' button at the bottom left.

## DREAMMAPPER – GENERATE REPORT

3. Enter today's date (or the last date when you have available data) and then select the report period, either 16, 30, 60 or 90 days. Click **Generate Report**.

Summary Report ×

**End Date**

**Day Span**

 ▼

Generate Report

4. This will display some summary data you can scroll through. To download a full report click the **PDF** icon. Depending on your web browser you may be prompted to **Open** or **Save** the report, you can choose which option you prefer.

Summary Report ×

**End Date**

**Day Span**

 ▼

Generate Report

---

Compliance Summary

Date Range	15 September 2016 - 28 September 2016 (14 days)
Device Serial Numbers	J1905681899F7
Device Model Names	BiPAP AVAPS 30 (DreamStation)
Days with Device Usage	1 day
Days without Device Usage	13 days
Percent Days with Device Usage	7.1%

## ATTACHMENTS

None